**INSTRUCCIONES:**

**A. Dilute the reactive.**

1. Before diluting the concentrate reactive, make sure that the water to be diluted is alkaline. To check it, put one or two drops of it in 1/3 ounce of water. If the water turns out purple, it can be used.
2. Dilute the concentrated reactive according the amount to be used, keeping a relation of 1 in 8, it means that one part of reactive must be diluted in seven equal parts of bidistilled or distilled water preferably (for example: one mL of reactive is diluted with 7 mL of water).
3. With the measure cup use the required amount of reactive on the dispenser flask and complete with bidistilled or distilled water preferably, using the marks that are on the flask until complete the indicated relation.

**B. Collect the milk of each quarter individually.**

1. Discard first stream of milk. Do not use colostrum or final milk samples of the quarters.
2. The paddle has four compartments marked as A, B, C and D to identify the samples of milk of each quarter. Draw foremilk from each quarter (approximately 2.5 mL) into the corresponding cup of testing paddle that has to be clean.

**C. Discharge excess milk of each cup of testing paddle.**

1. Tilt testing paddle to discharge excess milk.
2. Remaining milk should be level with the inside line in the cup (2.5 mL).


**D. Add an equal amount of the reactive solution on each cup.**

1. Put the paddle on horizontal position until the milk is distributed uniformly on each cup.
2. Slowly add the reactive solution into each cup until mixture is even.
3. ¡Avoid bubbles!
4. The mixture of milk and reactive must be on equal parts.

**E. Rotary movement of the paddle cause the reaction.**

1. Gently rotate mixture in a horizontal position. Do not mix more than 10 seconds.
2. Reaction can be observed almost immediately during rotation. Check results soon. The seen reaction disappears in about 20 seconds. Thickening or gel formation indicates high somatic cell counts and likelihood of mastitis.
3. The reaction gets a visual qualification. As much gel is formed, more is the qualification. The results can be interpreted using the chart show below.
4. Contact your veterinarian with positive results and use for assistance in adopting a treatment strategy.

**F. The paddle must be washed after each test to avoid false positives or false negatives.**

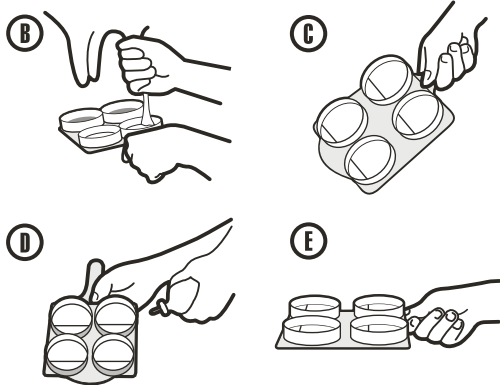**If you have questions or doubts related to MastiTest® Prueba de Mastitis del Dr. Calderón, please call to 511-4352323 or 511-7190105.**

DVM Company Resp.: Umberto Calderón O. CMWP 2944  
 Reg. SENASA Peru: B.02.702.N.0084; BOLIMA SENASA Reg. PUV- N° 007952/17;  
 Reg. Honduras: MW-2-983; Reg. Nicaragua: 1195; Reg. Panamá: RB-749-21.

MastiTest® is a registered trademark of Agrovet Market S.A.

Manufactured in Peru by Pharmadix Corp. S.A.C.  
 for and under license of Agrovet Market S.A.

Av. Canada 3792-3798, San Luis. Lima 15021 - Peru  
 (511) 2 300 300  
 ventas@agrovetmarket.com  
 www.agrovetmarket.com

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**MastiTest® CMT\***

Prueba de Mastitis del Dr. Calderón

\*California Mastitis Test developed by Schalm & Noorlander in 1957

Dilute before use - Concentrated Reactive

VETERINARY USE

**COMPOSITION**

Each 100 mL contains:  
Sodium lauryl sulfate 24.00 g  
Bromocresol purple 0.04 g  
Excipients q.s.ad. 100 mL

**What is MastiTest® Prueba de Mastitis del Dr. Calderón?**

MastiTest® Prueba de Mastitis del Dr. Calderón is a test that detects mastitis on a simple, fast, accurate and economic way; it is performed next to the animal to be evaluated, with an instant detectable reaction; being an effective method of monitoring changes on the udder health.

**What does the kit of MastiTest® Prueba de Mastitis del Dr. Calderón includes?**

1. One flask of 500 mL of concentrated reactive.
2. Test paddle designed for an accurate visual reaction.
3. Dispensing flask to administrate the reactive.
4. Measure cup.
5. Leaflet with the use instructions.

**REFILL:**

1. One flask of 500 mL of concentrated reactive.
2. Measure cup.
3. Leaflet with the use instructions.

**Routine**


The use of MastiTest® Prueba de Mastitis del Dr. Calderón at regular intervals (for example, every 2 weeks) and the record of results makes possible to detect mastitis in all stages and also to locate the affected quarter.

**Confirmed procedure**

MastiTest® Prueba de Mastitis del Dr. Calderón is a rapid detection test that detects immediately cases of mastitis in fresh milk or from individual quarters. The California Mastitis Test (CMT) was developed by Schalm & Noorlander in 1957 to estimate the content of leukocyte (somatic) cells of fresh milk. It's the best tool for early detection of dairy mastitis. This is a procedure proved that has the respect and approval of the dairy cattle industry.

**Call to your Veterinarian**

If the test shows a positive reaction to mastitis, as the chart shows, then you should call to your Veterinarian, who will know the required treatment for each animal with mastitis.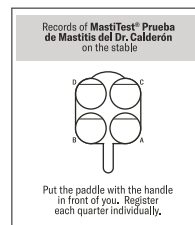

Identification of the Cow			
Test 1			
Test 2			
Test 3			
Test 4			

**Code acid/alkaline.**

- (+) Purple color/ alkaline milk
- (-) Yellow color/ acid milk

**Symbols of Mastitis**

- (-) Negative: Mixture remains liquid with no evidence of formation of a gel.
- (T) Traces: Mixture gets gelatinous.
- (1) Weak: the mixture forms a evident precipitate.
- (2) Confirmed: Mixture thickens immediately and forms a gel.
- (3) Strong: The mixture forms gel in the center, even when the movement on the paddle stops.

	Negative	Suspicious	Positive	Positive
<b>MastiTest CMT</b> Prueba de Mastitis del Dr. Calderón	Mixture remains liquid with no evidence of thickening or formation of a precipitate	Mixture gets viscose or similar to a gel.	Mixture is a evident gel.	A distinct gel forms which tends to adhere to the bottom of the paddle and during swirling a distinct mass forms.
<b>SAMPLE OF INDIVIDUAL QUARTER</b>	No Mastitis	Mastitis Traces	Mastitis	Mastitis
<b>SAMPLE OF MILK ON LARGER QUANTITIES</b>	No Mastitis	Confirmed Mastitis in one or more quarters.	Confirmed Mastitis check the quarters	Serious Mastitis check the quarters